
Treat-Eezi Rise/Recline Chair Overlay


FITTING INSTRUCTIONS

Please read carefully and follow the instructions to avoid damaging the product and its efficiency.



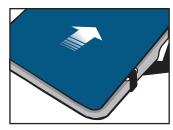

Recline the chair slightly so that the footrest is in an upright position. Lay the overlay on the chair in line with the end of the footrest. Using the strap provided secure the overlay around the back of the footrest and tighten if necessary.

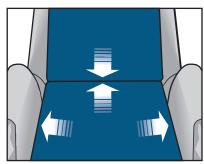

Once in position gently work your way up towards the seat without stretching the fabric.

Gently tuck any excess material down each side between the seat pad and armrest making sure the area of the seat and overlay are flat. Gently tuck a minimum of 10cm (4inches) of the overlay into the space between the seat and backrest, taking care not to stretch the fabric, this allows movement when the chair is tilted.



Working upwards gently flatten out the fabric and allow any excess length to overhang the top of the chair. Secure the strap around the back of the chair and tighten if required.

CLASS 1 MEDICAL DEVICE - REUSABLE

Suitable for clients at high risk of pressure damage who require full body length pressure distribution whilst sitting/reclining in their standard recliner chair. Please note there is no lower body weight limit on this product...Can also be used on any horizontal foam surface 10cm (4 inches) thick.

CONTRAINDICATIONS	SPECIFICATIONS
RISK OF SLIPPING DO NOT STAND ON THIS PRODUCT	Code: TEM60
NEVER STRETCH THE FABRIC	Length:195cm Width: 60cm approx.
ALWAYS REMOVE STRAPS/BUCKLES BEFORE WASHING	Max. User Weight 25st
ENSURE BOTH STRAPS ARE SECURE BEFORE SITTING	Material 100% Polyester

C1 South Ltd 1/7 Newton Place, Hamilton 3204

Phone: 07 8477488 email: info@c1south.co.nz web: www.c1south.co.nz